



PACKING INSTRUCTIONS “DIAGNA PAK AMB” SPECIMEN SHIPPER

FOR DIAGNOSTIC/BIOLOGICAL SUBSTANCE-CATEGORY B SPECIMENS

IMPORTANT WARNING: This packing instruction details correct assembly of this packaging design. Deviation may result in non-compliance and voiding of the certification. ICC The Compliance Center Inc. is not responsible for any packagings using substitute components or assembled incorrectly. If further information is required, please contact ICC The Compliance Center Inc. at 1-800-767-7231

SHIPPERS RESPONSIBILITY: includes insuring that all inner containers, closure, liners, cushioning, and (absorbent when applicable) will not react dangerously with the contents, and for AIR SHIPMENT, that inner containers for liquids are capable of meeting a minimum internal pressure differential of 95 kpa without leakage. Some volatile liquids may require a higher test pressure. See ICAO Technical Instruction, Part 3, Chapter 1, Sec 1.1.6.1 or IATA DGR 5.0.2.

PACKAGING COMPONENTS: 25 SETS/CASE (UNASSEMBLED)

1. Preprinted Outer Box, tuck & fold assembly (7” x 11-3/4” x 3-1/2”), packed flat
2. 95 KPA, 7 mil Absorbent Lined Specimen Bag (7-1/2” x 11”), printed w/Biohazard Symbol.

Packaging Assembly Instructions

1. Remove a preprinted flat box from the carton.
2. Take the (2) square flaps of the box, bend them forward, and fold in the side walls in.
Bend the back wall side of the box forward flush to the square flaps, fold the side walls to fit inside the lid sidewalls and bend forward to interlock.
3. Place your primary vials in cushioning material into the specimen bag.
With the bag flap reverse folded, roll your fingers upward to release the liner.
4. Lay the bag on a flat surface, fold the adhesive flap over onto the body of the bag to seal it.
5. Be sure to firmly press down on the entire flap area to adequately seal the adhesive flap to the body of the bag.
6. Insert the specimen bag diagonally (corner to corner) in the folded box, so as to suspend the sample in the middle space of the box.
7. Fold the box flaps inward & close. Secure the box flap with a sealing tape.
8. All required markings on this package are preprinted. No additional labeling is required.