

PK-INF250 Packing Instructions (Large)

BIOLOGICAL CATEGORY B SPECIMENS

Shipper must ensure that content is compatible with all packaging materials. For air shipment of liquids, Inner containers must be capable of meeting pressure requirements. IATA DGR 5.0.2.12.3 requires the use of a leak-proof liner for all liquid shipments. Deviations from any part of this Packing Instruction may void the UN Certification. Please consult 49 CFR for record retention requirement. If further information is required, contact ICC The Compliance Center Inc at 1-888-442-9628.

Packaging Components:

1. BX-106A White Fiberboard Box -11 1/4"x 6 5/8"x 3 1/2"
2. B14755PE Secondary Bag (9"x 12") with Biohazard Marking
3. BIOBAG1 Primary Bag-95 kPa self sealing bag (6"x 8 1/2")
4. INFSTRIP Absorbent Strip 4 1/2"x 2 1/4"



Assembly Instructions:

1. Place specimens in Primary Bag along with absorbent strip and seal bag.
2. Insert the Primary Bag into the 95 kPa envelope and follow closing instructions.
3. Write any contents and any special instructions on 95 kPa envelope.
4. Place entire envelope into White Box and close box.
5. Follow box closing instructions illustrated below.(1-7)

1



2




3




4



5

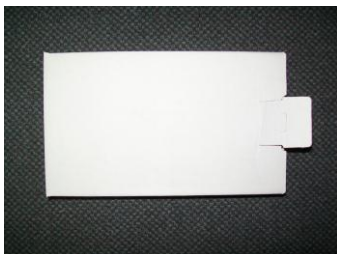

6

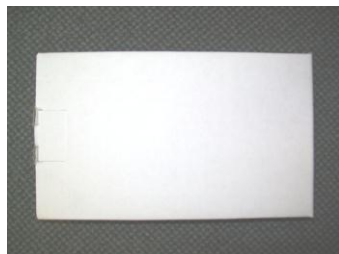

7

