

PK- INFPAK1 Packing Instructions

(PRE-ASSEMBLED KIT)
“INFEKTA PAK 1 (SINGLE)”



4G/CLASS 6.2/**
USA/+AX5518
**(DOM)

Shipper must ensure that content is compatible with all packaging materials. For air shipment of liquids, containers must be capable of meeting pressure requirements. IATA DGR 5.0.2.12.3 requires the use of a leak-proof liner for all liquid shipments. Deviations from any part of this Packing Instruction may void the UN certification. Please consult 49 CFR for record retention requirement. See current IATA packing instructions (Class 6) for toxic and infectious packaging. If further information is required, contact ICC The Compliance Center Inc at 1-888-442-9628.

Packaging Components:

1. Printed Outer Box (4-1/4” x 4-1/4” x 7-1/4”) (1)
2. Corrugated “B” perimeter Insert (1)
3. 95 KPA, 1-pint (2-1/2” x 6-1/4”) HDPE Screw-Cap Bottles w/Lined Caps (1)
4. 5-Hole Foam Insert (OPTIONAL) (1)
5. (4) White fiberboard Screw-Cap mailing tubes, seamed bottom & metal lids (3-1/8” x 6-3/8”)(1)

Assembly Instructions:

1. Remove Mailing Tubes/Plastic Bottles from the kit.
2. Remove the plastic bottles from inside the mailing tubes
3. Remove cap from plastic bottle.
4. Place primary vials in cushioning material into the plastic bottle.
Insert absorbent with the primaries
(Optional) Insert 5-hole foam inserts into the plastic bottle.
Place primaries in the cutouts of the foam inserts-add absorbent.
5. Tightly secure the lined caps onto the plastic bottle (70 lbs torque recommended).
Adding a strip of tape around the caps is suggested to insure security of the specimen.
6. Place the plastic bottle inside the mailing tubes & secure the metal lids on the tubes.
7. Insert the mailing tube w/bottle into the preprinted Infekta Pak 1 box.
Secure the top flaps of the box with a sealing tape.
8. **A SHIPPERS DECLARATION FORM IS REQUIRED WITH THIS PACKAGE!**
Attach the form on the blank side of the box with an adhesive envelope.