

## PK- INFPAK2 Packing Instructions

(PRE-ASSEMBLED KIT)  
**“INFEKTA PAK 2”**



**4G/CLASS 6.2/\*\***  
**USA/+AX5520**

\*\* (DOM)

Shipper must ensure that content is compatible with all packaging materials. For air shipment of liquids, Inner containers must be capable of meeting pressure requirements. IATA DGR 5.0.2.12.3 requires the use of a leak-proof liner for all liquid shipments. Deviations from any part of this Packing Instruction may void the UN Certification. Please consult 49 CFR for record retention requirement. See current IATA packing instructions (Class 6) for toxic and infectious packaging. If further information is required, contact ICC The Compliance Center Inc at 1-888-442-9628.

### **Packaging Components:**

1. Printed Outer Box (4-1/4” x 4-1/4” x 7-1/4”) (1)
2. Corrugated “B” flute locking finger Insert (1)
3. 95 KPA, 5.5 oz. (1-3/4” x 4-3/4”) HDPE Screw-Cap Bottle w/Lined Cap (1)
4. White fiberboard Screw-Cap mailing tube, seamed bottom & metal lid (2” x 5”) (1)

### **Assembly Instructions:**

1. Remove Mailing Tube/Plastic Bottle from the kit.
2. Remove the plastic bottle from inside the mailing tube
3. Remove cap from plastic bottle.
4. Place primary vials in cushioning material into the plastic bottle.  
Insert absorbent with the primaries.
5. Tightly secure the lined cap onto the plastic bottle (70 lbs torque recommended)  
Adding a strip of tape around the cap is suggested to insure security of the specimen.
6. Place the plastic bottle inside the mailing tube & secure the metal lid on the tube
7. Insert the mailing tube w/bottle into the finger inserts, then into the preprinted Infekta Pak 2 box.  
Secure the top & bottom flaps of the box with a sealing tape.
8. **A SHIPPERS DECLARATION FORM IS REQUIRED WITH THIS PACKAGE!**  
Attach the form on the blank side of the box with an adhesive envelope.