

## PK-INF250B Packing Instruction LARGE

### BIOLOGICAL CATEGORY B SPECIMENS

Shipper must ensure compatibility with all packaging materials and follow all appropriate transport regulations. For air shipment of liquids, containers must meet pressure, secondary closure and leak-proof liner requirements (**See IATA DGR Section 5 / ICAO TI Section 4 / US 49 CFR 173.27**). Deviations from any part of this instruction may void the UN certification. Please consult 49 CFR for United States record retention requirement. If further information is required, contact ICC Compliance Center at 1-888-442-9628.

#### PACKAGING COMPONENTS:

1. **BX-106A** White die-cut box 11 1/4" x 6 5/8" x 3 1/2" BIO (1)
2. **BI-475PE** Leakproof Ziplock Bag 7" x 8" (1)
3. **LB-BI475PE** Biohazard Trilingual Label for BI-4754/55 bags (1)
4. **BI- BIOBAG1** 6" X 8 1/2" 95kPa Biohazard Bag, tape closure (1)
5. **AB-INFSTRIP** Biosorb Strips 4.25" x 2.25" 100/pk (1)
6. **BI-INF600** 6 Bubble Wrap Pouches



#### Packaging Assembly Instructions

1. Place specimens in Primary Bag along with absorbent strip and seal bag.
2. Insert the Primary Bag into the 95 kPa envelope and follow closing instructions.
3. Write any contents and any special instructions on 95 kPa envelope.
4. Place entire envelope into White Box and close box.
5. Follow box closing instructions illustrated below.(1-7)






1

2

3

4

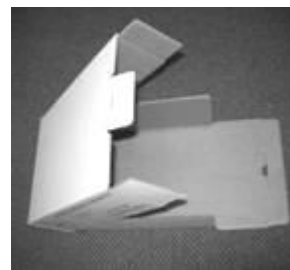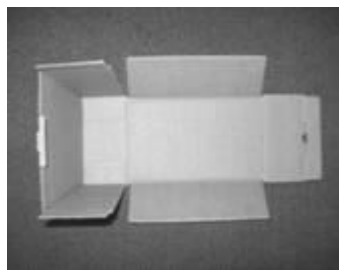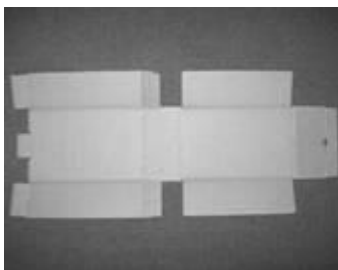

5

6

7

